ART. 700

mm Hg 18-24

CALZA AUTOREGGENTE A-G con tassello
STAY-UP STOCKING A-G with gusset

ART. 700-S

mm Hg 18-24

CALZA AUTOREGGENTE A-G
STAY-UP STOCKING A-G

ART. 702

mm Hg 18-24

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How to choose the size:

2 measurements are sufficient to choose the exact size: the ankle circumference (it is to choose the width S-M-L-XL) and the height of the person (to choose the length SHORT-MEDIUM-STANDARD); then compare the measurements with those in "Sizes table" on the front. The size table also reports the calf and thigh measurements.

Indications:

Pre-operative and post-operative embolism prophylaxis. For patients subjected to post-traumatic treatments and in case of hydrostatic oedemas, contusions, sprains.

Contraindications:

It is not advised to use in case open ulcers, dermatitis, erythemas, necrosis and in case the Doctor doesn't consider suitable its use. In cases intolerance to polyamide it can cause itch and irritation. It is not advised also wrong sizes.

Technical characteristics:

Gradual compression: maximum at the ankle level, progressively decreasing towards the calf and thigh, it restores compression lacking to the venous system (venous pump), re-establishing blood circulation rate to normal level in the leg's veins.

The product comes in various widths (small, medium, large, extralarge) and lengths (short, standard, long).

Each model has a different coloured tip, which allows easier identification of the model suitable for the various requirements.

In particular KK6 ANTI-TROMBOSIS ELASTIC STOCKING has certain high-quality features:

- Elastomer yarn: which is spirally covered with a protective sheath.
- Open toe: this feature enables the Physician to examine the foot more freely.
- Mesh: with circumferential stretch (alternately, double mesh) which allows the stocking to maintain its compression characteristics after several washings. The stockings has passed the quality and duration test (guarantee for 15 washes in washing machine at 95°C) maintaining the technical characteristics unaltered.
- Heel with anatomical shaping.
- Elastic band sewn to the top (antislip strip) ensures correct position of stocking and prevent descent.
- Mesh and packaging LATEX FREE.
- Material is resistant to the use of dermal creams, ointments and sweat.
- Possible washing and sterilization in autoclave (121°C).

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THROMBUS-EMBOLIC ILLNESS THROMBOSIS OF DEEP SEATED VEINS: PULMONARY EMBOLISM

Presence of blood clots or thrombi in the deep venous system of the lower limbs is of particular concern to Doctors. In fact, if a thrombus or part of one becomes detached and migrates through the systemic venous system (that is, becomes an embolus) it can become trapped in lung vessels, provoking an occlusion with even lethal results (see pulmonary embolism etc.).

In day-to-day movement, the activity of deep seated veins is constantly stimulated by the contraction of the calf muscles. This venous/muscular pump favours a correct blood flow and therefore reduces the risk of stagnation and the formation of clots. Nevertheless, in particular circumstances, this natural function of the organism comes to be missing. This is the case of persons who are bedridden after surgery (post-surgery confinement). These patients suffer from relaxation of leg muscles, deceleration of the cardiac pump and a marked reduction of blood-flow rate (stasis). In these cases the viscosity of the blood increases, above all at valve level of deep seated veins in the lower limbs.

To reduce this danger, modern surgery has found the solution which, for its simple use and proven efficiency, and especially its reasonable cost, represents an ideal answer to the problem: THE ANTI-EMBOLISM STOCKING.

POLYAMMIDE 75%
ELASTAM 25%

LATEX
FREE