COMPANY  
WITH QUALITY MANAGEMENT  
SYSTEM CERTIFIED BY DNV  
= ISO 9001 : 2008 =

COMPANY WITH QUALITY MANAGEMENT  
SYSTEM CERTIFIED BY DNV  
= ISO 13485:2003 =

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


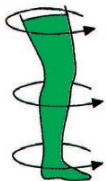
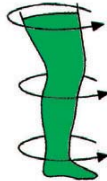
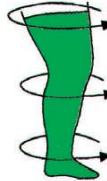
**anti-embolism stocking**

Prophylactic elastic stocking, decreasing compression

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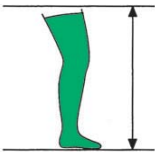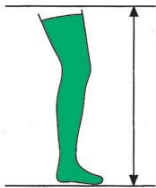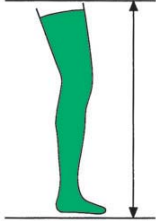#### How to choose the size:

2 measurements are sufficient to choose the exact size: the ankle circumference (it is to choose the width S-M-L-XL) and the height of the person (to choose the length SHORT-STANDARD-LONG); then compare the measurements with those in "Sizes table" on the front. The size table also report the calf and thigh measurements.

#### Indications:

Pre-operative and post-operative embolism prophylaxis. For patients subjected to post-traumatic treatments and in case of hydrostatic oedemas, contusions, sprains.

#### Contraindications:

It is not advised to use in case open ulcers, dermatitis, erythemas, necrosis and in case the Doctor doesn't consider suitable its use. In cases intolerance to polyamide it can cause itch and irritation. It is not advised also wrong sizes.

#### Technical characteristics:

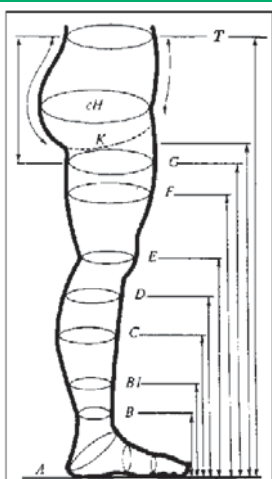
Gradual compression: mm Hg 18-24 at the ankle level, progressively decreasing towards the calf and thigh, it restores compression lacking to the venous system (venous pump), re-establishing blood circulation rate to normal level in the leg's veins.

The product comes in various widths (small, medium, large) and lengths (short, standard, long).

Each model has a different coloured tip, which allows easier identification of the model suitable for the various requirements.

In particular KK6 ANTI-TROMBOSIS ELASTIC STOCKING has certain high-quality features:


- Elastomer yarn: which is spirally covered with a protective sheath.
- Open toe: this feature enables the Physician to examine the foot more freely.
- Mesh: with circumferential stretch (alternately, double mesh) which allows the stocking to maintain its compression characteristics after several washings. The stockings has passed the quality and duration test (guarantee for 15 washes in washing machine at 95°C) maintaining the technical characteristics unaltered.
- Heel with anatomical shaping.
- Elastic band sewn to the top (antislip strip) ensures correct position of stocking and prevent descent.
- Mesh and packaging LATEX FREE.
- Material is resistant to the use of dermal creams, ointments and sweat.
- Possible washing and sterilization in autoclave (121°C).
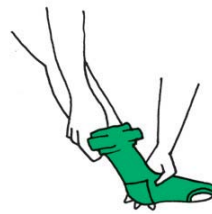
#### COME INDOSSARE LA CALZA - THE MANNER TO WEAR THE STOCKING

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